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- AGC Cem
- REF 8526 1 Doppelspritze à 5 ml, 10mg
- 12 Mischkanülen

**Product description**

AGC Cem is a self-curing, radio-opaque luting cement, specially developed for the permanent luting of secondary parts (double crowns) and bridges in luting structures (castings).

**Indications**  
• Extensive tooth defects: tooth decay, severe wear, traumatic injuries of the dental enamel  
• Tooth loss

**Contraindications**  
Do not use the material where there are contact allergies or known allergies to any of the components.

**Intended use**

Permanent adhesive cementation of crowns, bridges, inlays and onlays made of metal, ceramic or composite.  
Permanent adhesive luting of electroformed caps to luting structures or attachments of telescopic elements on model castings

**Limitations of use**

AGC Cem should always be used with a bonding agent (e.g. Luxabond universal for teeth or a suitable metal primer for dentures etc.). Please follow the manufacturer's instructions for use for the bonding agent used.

**Intended users**  
Dentist, dental technician

**Patient target group**

Persons receiving treatment as part of a dental procedure.

**Recommended use**

AGC Cem should always be used with a bonding agent (e.g. Luxabond universal for teeth or a suitable metal primer for dentures etc.). Please follow the manufacturer's instructions for use for the bonding agent used.  
Once AGC Cem has been expressed from the double-barreled syringe, the following times apply (the working and setting times depend on the ambient temperature):  
• Working time (including mixing time): max. 90 s min. (room temperature)  
• Intraoral curing time (including mixing time): min. 360 s (37 °C)  
• Extraral curing time (including mixing time): min. 15 min (room temperature)

**Application on patient**

- Remove the temporary restoration and carefully clean off any residual tem-porary luting cement from the cavity or prepared stump. Clean with water spray.
- Now the accurate fit of the restoration can be checked.
- Then clean the restoration thoroughly and pretreat as indicated by the manufacturer.
- To ensure a dry environment for the luting procedure with composite, the use of a rubber dam is recommended.
- Clean the cavity / prepared area using water spray and dry using air or pellets. Avoid either excessive residual moisture or over-drying.
- Apply the bonding agent (e.g. Luxabond universal) in accordance with the instructions for use.
- Apply AGC Cem directly into the restoration, using the tip. The working time for AGC Cem is max. 90 s. Fully position the restoration using gentle pressure. Allow the patient to slowly adopt the habitual occlusion. Cotton rolls are useful for this. Remove any excess material as described under: Removal of excess material. After 360 s, the restoration has fully gained its capacity to withstand stress.

**Application for metal work**

- Sandblast the surfaces to be bonded with aluminum oxide, ensuring that the surfaces are dust and oil free.
- Apply and work with the metal bonding agent according to the manufacturer's instructions for use.
- Apply AGC Cem directly onto the surfaces to be bonded using the tip. The working time is max. 90 s. Press the elements to be bonded firmly together until they are in the desired position. Remove any excess material as described under: Removal of excess material.
- After approx. 15 min the AGC Cem is completely chemically cured and further processing can begin.

**Removal of excess material**

Excess material should be removed during the elastic phase, i.e. approx. 30 - 60 s after application, using a single-use microbrush / brush / foam pellet / dental floss or scaler. It is particularly important to ensure that excess is removed quickly in areas that are difficult to access (proximal, gingival margins).

**Interactions**

Do not use AGC Cem together with materials that contain eugenol. We therefore recommend eugenol-free temporary cements.

**Using the double-barreled syringe**

The double-barreled syringe enables automatic dispensing and mixing of the material, which allows a time-saving direct application. When using a new double-barreled syringe, remove the transport cap first. The mixing tip on the double-barreled syringe acts as a seal. The used mixing tip is replaced with a new tip when the device is used for the next treatment.

Note: When using a double-barreled syringe for the first time, discard a small amount of the initially ejected material (e.g. a pea-sized amount).

**Residual risks / side effects**

There are no known side effects to date. The residual risk of hypersensitivity to components in the material cannot be ruled out.

**Warnings / precautions**

- Keep out of the reach of children!
- Avoid contact with the skin! In the event of accidental skin contact, immediately wash the affected area thoroughly with soap and water.
- Avoid eye contact! In the event of accidental contact with the eyes, immediately rinse thoroughly with plenty of water and consult a physician if necessary.
- Follow the manufacturer's instructions for other products that are used with the material/materials.
- Serious incidents involving this product must be reported to the manufacturer and to the responsible registration authorities.

**Performance characteristics of the device**

The product complies with ISO 4049:2019, Class 1, Type 2 (including colour stability)

**Storage / Disposal**

- Store at 2 - 25 °C (36 - 77 °F) in a dry place!
- Protect against exposure to direct sunlight!
- After application, leave the used mixing tip on the double-barreled syringe as a seal!
- Do not use after the expiration date!
- Disposal must comply with national regulations.

**Composition**

Dental glass, bis-GMA, EBPADMA, TEDMA, TMPTMA, MMHE, SiO<sub>2</sub>, dithio-, inorganic filling materials; approx. 49 vol-% (0,02 - 4µm).

**Trading forms**

- 1 double syringe x 5 ml, honey
- 12 mixing tips